



**DEVOLPE**  
MEDICAL



## INSTRUCTIONS FOR USE

### ESCACHAIR



## 1. Information on safety

### Warning







- ✚ Untrained operators can cause injury or be injured. Permit only trained personnel to operate the chair.
- ✚ Improper use of the chair can cause injury. Use the chair only for the purpose described in this manual.
- ✚ Improper operation can cause injury. Operate the chair only as described in this manual.
- ✚ An unattended patient can be injured. Stay with the patient at all times.
- ✚ An unrestrained patient can fall off the chair and be injured. Use restraints to secure the patient on the chair.
- ✚ Helpers can cause injury or be injured.




**DEVOLPE**  
MEDICAL

- ✚ Operate and maintain control of the chair and direct all helpers.
- ✚ Improper maintenance can cause injury.
- ✚ Maintain the chair only as described in this manual.
- ✚ Improper parts and service can cause injury.
- ✚ Use only Devolpe parts and Devolpe-approved service on the chair.
- ✚ Modifying the chair can cause injury and damage. Use the stretcher only as designed by Devolpe.
- ✚ Worn or damaged restraints can fail and cause injury. Inspect restraints regularly. Dispose of restraints that are worn, damaged, or have been involved in a vehicular accident.
- ✚ Modifying the chair can cause unpredictable operation and injury. Do not alter or modify the chair to fit the ambulance.
- ✚ Never exceed load capacity of the device.
- ✚ Maintain control of the chair at all times.
- ✚ Attaching improper items to the stretcher can cause injury. Use only Devolpe -approved items on the stretcher.

## Instruction labels

 <b>Read IFU</b>	 <b>Load capacity</b>	 <b>Be cautious of fingers</b>	 <b>This way up &amp; Keep dry</b>
 <b>Keep away from fire</b>	 <b>Only use trolley</b>		

 <b>Press</b>	 <b>Push the tracking belts releaser</b>
--	--



## 2. Skills and trainings

Operators using the stretcher need:

- a working knowledge of emergency patient handling procedures.
- the ability to assist the patient.
- a complete understanding of the procedures described in this manual.

Operators shall pass training set by their training officer in accordance with training program.

## 3. About the chair

Escachair with stair belts is a device used to transport patients to hospitals for further medical treatment in a sitting position.

The device is operated by minimum of two users.

Key peculiarities of the device:

- It has 220 kg load capacity;
- Sitting part can be made of a metal or aluminum alloy covered by fabric, depending on the order request;
- The device has two front 360° swivel wheels and two back wheels equipped with brakes;
- The device has pair of non-slip handles at the foot bar and top frame of the device.
- Footrest provides comfortness during transportation.

Features:

- The device paint is water, desinfectants and scratch resistant;
- The frame consists of sturdy lightweight non twisting construction that is free of edges;
- The device is equipped with the sets of 2 quick-release patient restraints;
- Lying part of the device is produced by antibacterial, antifungal, anti-stain and anti-pudrid material that is waterproof, easy to wash and clean all kinds of liquids & petrol-oil and to desinfect.

**Used materials: Aluminum, stainless steel, steel and their composites and fabric.**

### TECHNICAL CHARACTERISTICS

Product	Catalogue code	Length (cm)	Width (cm)	Height (cm)	Dimensions of the rear wheels (mm)	Weight (kg)	Load capacity (kg)
EscaChair with stair belts	FPTC/01/ECDP	75 (non-folded) 35 (folded)	48	105	100	15	220

### Vehicle considerations

The ambulance patient compartment must have:

- A smooth, level floor large enough for the stretcher.
- An approved, installed Devolpe chair fastener configured for hang the chair inside ambulance.



**DEVOLPE**  
MEDICAL



Tracking belt  
releasing lever

- Use the tracking belt releasing lever to unfold the stair belts to be used during stairs.



Folding hinges

- Use the Folding hinges to fold and by vice versa movement to unfold the chair.



**DEVOLPE**  
MEDICAL



Use the red  
lever to  
extend the  
main  
handgrips

- Use the handgrip red levers to extend the handgrip into 3 preferred positions.

\*Note: please contact the manufacturer for more info.